sanowell

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Safety information. Keep the manual for subsequent use!

- Please read the manual carefully before using your air bubble foot bath for the first time.
- Do not immerse the device in water or any other liquids and do always touch the switch, cable, and the plug with dry hands only.
- In case any liquid should ever have come into the casing, immediately unplug the device.
- Never use the device if the cable or the plug are damaged, and keep the cable from heated surfaces.
- Use the device with suitable bath additives and water only, never with any other liquids.
- Turn of the device after use and unplug it from the electrical outlet.
- The device is not suited for use under full body weight. Therefore never use the device while standing.
- For use set the device on a firm floor surface to avoid tilting.
- Do not use the cable for carrying or pulling purposes.
- Take special care for orderly use of the device, if used in the vicinity of children or disabled persons.
- Do only apply according to the operation manual and its destined purpose.
- The voltage of the device has to be in conformity with the countries common voltage.
- Do not leave the device unobserved while turned on.
- Do not use the device outdoors or in humid areas.
- Do not cover up the air vents on the bottom of the device.
- In case of disturbances discontinue use. Consult your dealership and have the device repaired by authorised service agents only.
- In case you suffer from foot problems, vein disease or diabetes, or if medical problems arise during use, please consult your physician.
- If water leaks from the appliance, the appliance must no longer be used.

Delivery contents

Please check first if your sanowell PlusTherm has been delivered completely as described. The contents of delivery is as follows:

- 1 x Air bubble foot bath sanowell PlusTherm
- 4 x Massage and acupressure accessories
- 1 x operation manual

Congratulations

Every day our feet are highly strained. Despite we do not show them the necessary attention, until they are very tired or even hurt. With the purchase of the sanowell air bubble foot bath PlusTherm you can actively contribute to the care and health maintenance of your feet. Same as all other sanowell devices, your air bubble foot bath has been checked by a certified proofing agency. In order to use the variable application methods effectively, we ask you to please read the operation manual with care.

Care of your feet with the sanowell PlusTherm

Care of feet is much more than simply a question of aesthetics, as well cared for feet do not only look good, but are usually healthier and therefore more capable of bearing. Most of the time a few minutes a day are enough to give your feet a better look and vitality. One very important part of foot care is the relaxing foot bath. The sanowell PlusTherm offers the best supposition for that. It enables foot massage with or without water, intensive massage of reflex zones, as well as radiant warmth exposure by use of ascending warmth. According to personal preference the setting of the device can be chosen to massage or air bubble function alone or in combination with ascending warmth. Independent of the setting you may choose, you will receive a beneficial relaxing result for body and soul.

Massage

The sanowell PlusTherm is equipped with vibrating nodes, air bubble bar, ascending warmth and exchangeable massage accessories, and it gives you thus a foot massage on the soles as well as the top of your feet. The foot massage contributes to a better blood flow in the surrounding tissue, so that veins and lymph passages are decongested. Tense muscles are relaxed and the entire metabolism is stimulated.

Warmth-treatment

The sanowell PlusTherm heats the water in the device up to 40 °C, i.e. ensures during the entire application time a constant temperature and prevents thus a fast cooling of the water.

Magnetic therapy

Ideal in the instance of oversteering or for pain subsidence with heel or other foot problems are the 16 inlayed magnets in the bottom of the tub. They stimulate blood circulation and provide you thus with more energy.

Treatment time

You may take about 10 to 30 minutes time each day for your foot bath with massage. Please do observe though, that before renewed operation the device has to be completely cooled off.

Medical recommendations

As each massage is individually sensed, it depends on your preference which application option offered by the sanowell PlusTherm is the most comfortable to you. You may try whether a soft or intensive stimulation brings you optimum relaxation. In case of medical questions concerning the application, your physician will most likely be glad to be of help.

In the case of inexplicable pain or swelling in the legs or feet, as well as after a muscle injury, you should consult your physician before application. Discontinue a massage if during application pain or swelling occur. During pregnancy it is advised to consult your physician about application.

Operation

Foot bath and massage with water

Before filling water into the tub, make sure that the device is not connected to a electrical source, i.e. you may not connect the cable to an electrical plug until the air bubble foot bath is filled with water and stands securely on the ground! Do not place towels, carpets, clothing or similar items under the unit, as this would block the air slots.

Foot massage without water

You may use the device for a foot massage without water – the setting switch needs to be turned to position M (vibrating massage). Heat sensitive persons should put on socks when using the sanowell PlusTherm without water.

Attention:

If the air nozzles on the bottom of the air bubble foot bath are plugged, the motor may be damaged. Therefore, please do not use foils or other materials to cover them and do not stand on the nozzles during use. We recommend the sitting position when using the foot spa. Warranty will expire in case of noncompliance.

Application

Filling the device with water

The sanowell PlusTherm can be filled with up to 4 litre of water. The maximum fill-height is marked on the inside of the air bubble foot bath. You may use cold or warm water with or without non foaming bath additives.

Bath additives

To make your therapy even more comfortable, you can use special bath additives that are suitable for foot spas, like our sanowell herbal baths. Please do not use bath salts or any other non-fluid additives (e.g. alkaline foot baths), as these might block the air nozzles. In case the device has been damaged that way the guarantee will expire.

Setting the function

Sit down comfortably in front of the air bubble foot bath and choose one of the three different functions by turning the switch.

Choosing warmth

The sanowell PlusTherm is equipped with an innovative warmth function. This new technique ensures that during an application the water stays constantly at the same temperature according to setting or heats the water up to 40 °C.

Position M

Foot massage with or without water. In this position of setting your feet are gently massaged by about 400 vibrating nodes. You may use this function either with or without water. Please observe, that on this setting the water, if used, will not be heated up, but merely prevented from cooling down.

Position W+A

Air bubble foot bath with ascending warmth. If you wish to enjoy a combination of air bubble foot bath and ascending warmth, turn the switch to this position. In the position W+A additional to the air bubble foot bath the warmth function is activated. Cold tap water will be slowly heated up to 40 °C in a time span of about 30 minutes. The temperature is kept during the entire application. If the water is too hot for your feeling at the position W+A, you can switch back to position M and wait until it has cooled down to the desired temperature. In case you want to cool down the water by filling up with cold water, please be careful not to exceed the maximum fill-height. This setting option is for use with water only.

Position M+A+W

Air bubble foot bath, vibrating massage and ascending warmth. This form of application combines the vibrating massage with the air bubble foot bath in the hot water. Set the switch to this position. Use only with water. In this function the water is slowly heated up to 40 °C. If the water you fill in has about 30 °C already, the bath water will be heated up to 40 °C in a matter of about 25 minutes.

Position 0

In this position the device is turned off.

Attention:

If your body reacts sensitively to warmth, you are prone to circulatory problems, swelling of the extremities, or you suffer from an infection, irritation or open sores of the skin, you should consult your physician before application.

Cleaning and care of your sanowell PlusTherm

Please always take care that for cleaning purposes the device is turned off and unplugged. Drain the water by letting it out of the draining spout and wipe the cooled down unit with a cloth. Please restrain from using aggressive substances like scrubbing powder or dish washing detergent in order to keep the surfaces from corrosion. Please store your PlusTherm at a dry and cool place. In order to keep from breakage, roll up the cable carefully.

Advice on recycling

The Packaging should be disposed of in the garbage recycling, the metal parts of the device in the used metal collection. Plastic parts, electronic pieces and conductors need to be disposed of as electronic garbage. For further information concerning disposal of problem waste contact your nearest recycling center.

Technical data

Current: 220 – 240 Volt/50 Hz

Power: 345 Watt

Operating conditions: 0° C up to + 50° C at 20 % to 75 % relative humidity, but without condensation, 700 hPa to 1060 hPa.

Storage conditions: 0° C up to + 50° C at 20 % to 75 % relative humidity, but without condensation, 700 hPa to 1060 hPa.

Measurements: about 42 cm (length) x 36 cm (width) x 19 cm (height)

Weight: about 3,1 kg (without massage accessories)

We reserve the right for changes to improve and further develop this product.

Last but not least

We wish you great success and relaxation with the sanowell PlusTherm and hope that for a long time to come you will enjoy the device and the feeling of having well cared for and relaxed feet. As one of the leading companies in the sector of wellness and home care, we would like to use the opportunity to point out our large line of products in the area of physiotherapy, alternative pain therapy and qualified body care. Please feel free to contact us any time you have questions about this or any other product.

We are always there for you...

hofmann gmbh
the wellness and care company

Warranty

There is a 2 year guarantee for our products, applicable from date of purchase.

Within the prescribed warranty time we will remove all defects of the device, that are based on faulty material or manufacture. Removal of defect occurs at our choice by either repair or replacement of parts or the entire device.

Exempt from warranty are: Defects caused by inappropriate use (f. ex. connection to inappropriate electrical current or voltage, connection to inappropriate sources of electricity, breakage by fall and such). Normal wear and tear that only minimally affects the operational utility of the unit is not subject to the warranty.

Any damage resulting from misuse – especially noncompliance with the operating and care instructions – will not be covered by warranty.

Warranty subsides if unauthorized intervention or repair is undertaken, or if non-original parts are used.

In the case of a warranty claim please forward the product with the original receipt to one of our service partners or directly to the hofmann gmbh.

No other claims are valid under the provisions of this warranty, unless our liability is based on legal statute. Claims against the dealer based on the provision of the purchase agreement do not affect this warranty.

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